



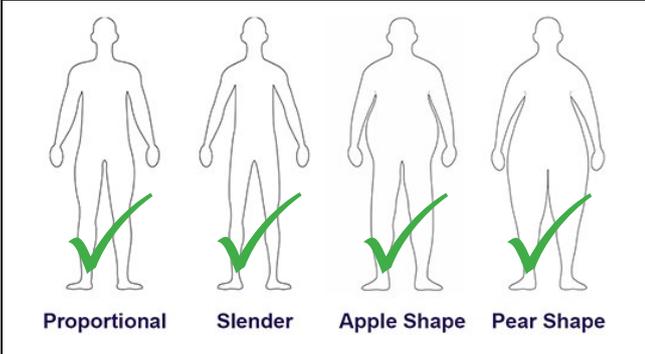


CAUTION: This document is intended as a guide only and in no way replaces an individual patient risk assessment. Patient specific conditions, functional independence measure (FIM) and weight distribution may influence the most suitable sling size as well as the most suitable type.

 <p>Proportional Slender Apple Shape Pear Shape</p>	<p>Body shape and size are important considerations when prescribing a sling. This quick reference tool is designed to help determine the compatibility of an individual with this type of sling.</p> <p>✓ = Good general compatibility. ✓ = May be suitable. ✗ = Probably not compatible. Use only with extreme caution.</p>
---	--

Key Dimension: Top of Head > Seat

Take the client measurement as shown in the diagram below and reference it against the corresponding measurements in the colour-coded chart to help determine the most suitable sling size.

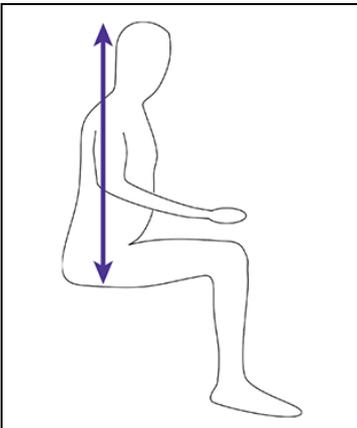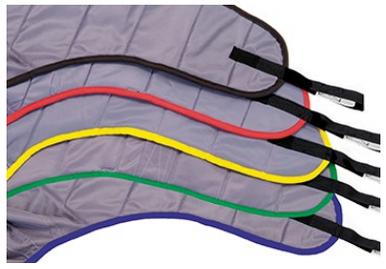	Sling Size	Dimensional Range		<p>Oxford slings have colour-coded binding matching this chart to help quickly identify the size.</p> 
		Min	Max	
	XS	670mm	750mm	
	S	730mm	850mm	
	M	820mm	960mm	
	L	930mm	1010mm	
XL	950mm	1100mm		

For further assistance, core dimensions for each sling size are provided on the next page.

Selecting a sling for an individual involves more than simply measuring their size. Other considerations, as part of a comprehensive risk assessment, should include, but are not limited to:


- Type of transfer (e.g. bed to chair, chair to toilet, etc)
- Level of support required
- Patient conditions
- Physical and cognitive ability of the patient
- Abilities and limitations of the carer
- Compatibility of the sling with other equipment
- Is a commode aperture required?
- Is the sling to be used for bathing?
- Potential obstacles and hazards during transfer



It is critical to assess all possible risks when planning to lift and transfer a patient from one surface to another. Failure to assess the risk correctly may result in serious injury to the patient and/or caregiver.

Core Sling Dimensions: Oxford Full Body

Full Back			
Sling Size	Dimension A	Dimension B	Dimension C
XS	910mm	840mm	1240mm
S	1000mm	970mm	1460mm
M	1110mm	1060mm	1580mm
L	1210mm	1200mm	1760mm
XL	1390mm	1300mm	1920mm

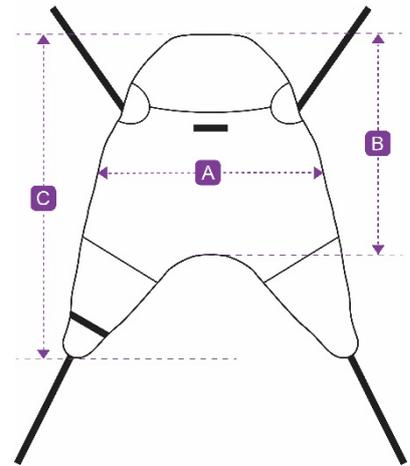

*All dimensions are subject to manufacturing tolerances.

Custom Sling Service:

Joerns Healthcare offer a custom sling service should you have any non-standard or patient specific requirements. Please contact Joerns Healthcare for more information and guidance. Extended lead times apply.